CLINICAL ASPECTS OF COMPACT GANTRY DESIGNS

J. Heese, J. Wulff, A. Winnebeck, A. Huggins, M. Schillo



VARIAN PARTICLE THERAPY JUERGEN HEESE

New gantry developments Viewpoint from user and vendor perspective

what is important for the customer?

- precise dosimetry system
 - minimum corrections
- simple and time efficient measurements to set up the clinical system
 - short clinical commissioning time



Clinical aspects of compact Gantry designs Agenda (focus on modulated scanning only)

- TPS setup for modulated scanning
 - required measurements
- Spot positions
- Spot shapes
- SAD
- Upstream scanning
- Achromatic optics



Modulated Scanning Commissioning of Treatment Planning System



4 | VARIAN PARTICLE THERAPY

VARJAN DE BEAM

Divergence Evolution of beam size around isocenter



Linear increase of sigma with distance from nozzle

Parametrized in treatment planning (linear/quadratic)



Spot position accuracy

- Significant deviations between planned and delivered spot position will cause dose inhomogeneities
- Scanning correction required



- A. Huggins, master thesis
- 6 | VARIAN PARTICLE THERAPY

VARIAN medical systems

Before Scanning Magnet Calibration



After Scanning Magnet calibration





- After calibration, typically sub-mm accuracy of delivered vs planned spot positions can be achieved (at least in VARIAN systems)
- Not dependent on gantry angle
- Standard procedures exist to calibrate scanner magnets and derive corrections



Spot shapes

- currently, all commercial treatment planning systems assume gaussian spot shapes
- new gantry designs might imply deviations from gaussian shapes
- most commercial treatment planning systems do not expect variations of the spot shape over the scan area
- If spots in air are non-gaussian, does multiple Coulomb scattering (MCS) help to reduce effects?





Spot shapes in current scanning system



Circular spots over full field size of 30cm x 40cm from 245 to 70 MeV

11 | VARIAN PARTICLE THERAPY



Spot shape characterization for treatment planning:

- one measurement per energy interval on central axis at isocenter required
- data can be significantly reduced: 2D gaussian fits, extraction of sigma-x, sigma-y per energy point
- repeated at typically 3 distances from isocenter and multiple gantry angles



Spot shapes on future system Where is the limit?



Dedicated TPS parametrization required:

Elliptical spots (if gaussian):

- need sigma-max, sigma-min and ellipse angle ٠
- All parameters required over full field size ٠
- Smooth variations of parameters required •



Could this spot shape be parametrized?



Multiple Coulomb Scattering Effect on spot size



14 | VARIAN PARTICLE THERAPY

VARIAN medical systems

Effects of MCS ... no fast reduction of sigma differences



15 | VARIAN PARTICLE THERAPY



- Deviations of spot shapes from gaussian need parametrization in TPS
- The more gaussian, the better
- MCS may not be sufficient to smooth out deviations (ellipticities or non gaussian shapes)
- In case of large deviations from gaussian shapes, a robust algorithm to determine spot centroids for scanner corrections is required







| VARIAN PARTICLE THERAPY

Clinical Aspects

- is a large SAD required for modulated scanning?
 - skin dose limitations as a driver?
 "the dose to the skin relative to the dose in the SOBP increases with decreasing effective source-to-axis distance (SAD)".
 - correct statement, but significant effect?
- does not consider
 - dose enhancement in Bragg peak
 - fluence modulation in modulated scanning
 - multiple fields
 - realistic tumor shapes and patient dimensions



 r_1

 \mathbf{r}_2

Proximal dose effects for small SAD

central axis depth dose for a monoenergetic homogeneous field (Eclipse calculation with ProBeam beam model for different SADs)



19 | VARIAN PARTICLE THERAPY

VARTAN

ProBEAM

Effects for small SAD

Phantom study with Eclipse, two opposed SFUD /IMPT fields



Proximal dose enhancement of 1.5m vs 2m SAD relative to target dose for SFUD optimization: 5.7% for IMPT optimization: 1.9%

20 | VARIAN PARTICLE THERAPY



What are the SAD effects in treatment planning? Phantom study with Eclipse, two opposed SFUD /IMPT fields



- field patching: can be handled by treatment planning independent of SAD
- spherical distal edge isodoses no clinical impact





 In the presented study no evidence for significant adverse effects of a short virtual SAD in modulated scanning down to 150cm has been identified





Upstream scanning Variable virtual SAD

- Virtual SAD could be variable (dependent on scanning deflection in bending direction)
- Typically small impact on dosimetry due to large virtual SADs
- Variable SAD is typically not considered in treatment planning system
- Could have big impact in case of short virtual SADs

23 | VARIAN PARTICLE THERAPY



Large momentum acceptance bending magnets

Example:

Consider a modulated scanning treatment plan which requires one layer at 180 MeV

- Case 1: E_{max} 200 MeV -> bending magnet field set to 200 MeV Layer energy 180 MeV with magnet field setting of 200 MeV
- Case 2 E_{max}=180 MeV-> bending magnet field set to 180 MeV layer energy 180 MeV with magnet field setting of 180 MeV
- Case 1 and case 2 could be two examples to produce a layer energy of 180 MeV, but there are many more as E_{max} is variable
- > leads to degeneracy of bending magnet current for a requested beam energy



VAR AN medical systems Large momentum acceptance bending magnets

- trajectories in bending magnet and optics for one energy could be different due to different magnetic fields
- Could lead to degenerated 2D correction function (e.g. scanner magnet calibration)
- variable virtual SAD effects in y, also effects due to different trajectories in x?
- Energy/range verification required (IEC 60601-2-64)



Impact of additional corrections Potential mitigations

CORRECTION TABLES IN TREATMENT MACHINE

• Additional commissioning work – cost driver

CORRECTION TABLES BY USER

- Special knowlegde and training for user required
- Additional responsibility for user
- Additional measurements for user
- ADDITIONAL CORRECTIONS IN TREATMENT PLANNING SYSTEM
- Additional work for user



Summary



- possible design could look like this:
 - Small bore superconducting bend (4T)
 - optimized downstream scanning, short SAD



- large virtual SAD is beneficial to minimize variable SAD-y
- otherwise, TPS calculations or machine corrections have to be implemented
- Spot shape variations to be evaluated
- Large momentum acceptance
 - Will have a more complicated impact. Solutions that work with minimum corrections have to be developed.



27 | VARIAN PARTICLE THERAPY

VAR AN medical systems

THANK YOU!

... questions?

